<b>EN</b>	English	1-22
<b>CN</b>	中文 (简体)	23-42
<b>FR</b>	France	43-62
<b>ES</b>	España	63-82
<b>IT</b>	Italia	83-102
<b>DE</b>	Deutschland	103-122

## Solax Technology Limited

### Symbol Definitions

The following symbols are found on the Scooter:

- |  |   |  |  |
|--|---|--|--|
|  | Manufacturer                                    |  | Item is recyclable   |
|  | Importer  |  | Warning. Non-ionizing radiation  |
|  | Authorised Representative of the European Union |  | Do not drive in rain or snowy weather  |
|  | Date of Manufacture                             |  | keep loose clothing clear of the scooter   |
|  | Refer to The Instruction Manual                 |  | Do not operate cell phones while the scooter is powered on.                              |
|  | Medical Device                                  |  | Warning. Beware of potential hazard  |
|  | Serial Number                                   |  | Warning. Pinch hazard  |
|  | Catalogue Number                                |  | WEEE Label. Do not discard the item in general waste. Follow the local recycling policy. |
|  | Batch Code                                      |  |  |
|  | CE Mark   |  |  |

## Solax Electric Scooter Instruction Manual

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**Dongguan Prestige Sporting Goods Co., Ltd**  
No.8,Road.1,3rd Ind. District, Qiaotou Area,  
Houjie Town, Dongguan City, Guangdong  
Province, China

**Share Info GmbH**  
Heerdter Lohweg 83, 40549 Düsseldorf, Germany

Issue 2020-09

# Introduction

## Dear Customer:

Thank you for choosing Solax-Scooter !

Please read this manual carefully before operating, because this manual will help you to operate the Mobie and Genie smoothly. This manual is applicable to the Mobie and Geniescooter. Due to the similarity place, please select your own model. If you can't understand all of the content completely or you need the assistance, please contact the scooter dealer of Solax scooter or Dongguan Prestige Sporting Goods Co. Ltd.

TEL : 86-769-88918228

E-mail:alex@solaxtech.com

This manual covered Mobie and Genie operational approach, equipment instructions and the methods of dealing with emergencie. Please note, except the feature description and function introduction are different, other content under unified use.

The following symbols used in this manual in order to indicate warning and matters need to be attention; therefore, it is important to you to fully understand and master the

**WARNING:** Failure to note the warnings from this manual will cause your personal injury.

**NOTIFICATION:** Failure to note the notification from this manual will damage your scooter.

All the information and pictures in the manual are determined after the scooter left the factory. If there were some changes on the designing, there will be no further notification. Please understand and support us on this. At last, we believe that SOLX-Scooter will not only bring comfortable and convenience on your movement, but also make you feel even more joyful about your life.

# Contents

◆	Feature Description	»»	-----3-5
◆	Function Introduction	»»	----- 5-13
	»» Mobie model		
	Control Panel	-----	5-6
	Power switch	-----	6
	Speed controller	-----	6
	Battery power indicator	-----	7
	Unfolding and folding body	-----	7-8
	Telescope handlebar	-----	9
	»» Geniemodel		
	Control Panel	-----	11
	Power switch	-----	12
	Speed controller	-----	12
	Battery power indicator	-----	13
	Unfolding and folding body	-----	13-15
	Telescope handlebar	-----	15
	Controller	-----	15
	Brake	-----	15
	Motor/ Transmission component	-----	15
◆	Specification	»»	----- 13
◆	Safety Requirements	»»	----- 14-15
◆	Battery and Battery Charging	»»	----- 15-18
◆	Operation Instruction	»»	----- 18
◆	Basic Troubleshooting	»»	----- 19
◆	Maintenance	»»	----- 19-21

### The product should not be use under the following situations:

1. Person who is sick or feeling uncomfortable.
2. Patient with acute tumor.
3. Patient with heart disease.
4. Women in pregnancy or during menstruation.
5. Children under 12 years old.
6. Person who worries about his own health should consult a physician before use.
7. Consult a physician before use who use the pacemaker.
8. Patient with action inconvenience or slow perception.
9. Person who is unconscious,with bad coma blood circulation or severe sensitive skin.

# Feature Description

## ◆ Feature description of scooter:

Scooter is mainly composed of the following parts: Front and rear frame, Control panel, Front/Rear cover, Front tires, Rear tires, Handle switch, Handle, Drawtube, Folding seat, Seat carriage, Folding handrail, Footboard, Carry handle, Battery, Charger, Motor set. (Mobie FIG-1, Genie FIG-2)

The overall design from Solax-Scooter is full of modern recreational flavor. Safety and comfort, easily operated, folded simply and conveniently carried, lightweight and stable. It mainly has following characteristics:

1. Frame is consisting of advanced aluminum alloy with light, strong and durable.
2. Don't need any tool to folding and unfolding. Easy operation.
3. Seat, front and rear frames can be folded, also easy to carry.
4. Scooter is a portable and foldable scooter with light weight and small size which can be put in the trunk or take it with you through the public transportation.
5. Battery is easy to load and unload, also convenient for user to carry.
6. Comfortable—fit the ergonomic seat; Handle bar can be adjusted to a comfortable height and gradient.
7. 6 inch solid front tires and 7 inch solid rear tires design.
8. Starting smooth and braking stability.
9. Body shape-stream is beautiful and elegant, surface is smooth and bright, environmental protection and durable.

- |                 |                   |                  |              |
|-----------------|-------------------|------------------|--------------|
| ① Control panel | ⑤ Decorative lamp | ⑨ Carry handle   | ⑬ Rear cover |
| ② Handle        | ⑥ Front cover     | ⑩ Rear frame     | ⑭ Seat frame |
| ③ Drawtube      | ⑦ Front tires     | ⑪ Rear tires     | ⑮ Seat       |
| ④ Handle switch | ⑧ Front frame     | ⑫ anti-tip wheel | ⑯ Armrest    |
|                 |                   |                  | ⑰ Backrest   |

- |                  |                 |                |              |
|------------------|-----------------|----------------|--------------|
| ① Control panel  | ⑤ Handle switch | ⑨ Carry handle | ⑬ Seat frame |
| ② Handle         | ⑥ Front cover   | ⑩ Rear Frame   | ⑭ Seat       |
| ③ Drawtube       | ⑦ Front tires   | ⑪ Rear tires   | ⑮ Armrest    |
| ④ Release switch | ⑧ Front frame   | ⑫ Rear frame   | ⑯ Backrest   |

In conclusion, Solax-Scooter is a unique scooter, for middle age, elderly, disabled, walking difficulties, weak and lack endurance elder people, cripple or physically disabled people. The scooter is a trust and worthy product. It also used for theme park, amusement park, zoo, museum etc. , meanwhile it is ideal ride instead of walking, shopping tools for middle aged and elderly people. (Mobie FIG.3 , Genie FIG.4)

## Function Introduction

### ◆ Mobie scooter

#### ››› Control panel

The control panel is an operating controlling component, which is mainly to control power supply's connection and cut off, moving forward and backward as well as show the battery level, including power switch, speed control lever , battery condition indicator, control panel, cover, etc.(Mobie FIG.5)

S2043 control panel FIG.5

**NOTIFICATION:** Do not put the control panel in the humid area. If the control panel has been affected by moisture before using, please ensure it is dry when you operate it.

#### ››› Power switch

Mobie is the Key switch(Power switch-6a and rocker switch-6b)

If the power switch design is the key switch, please insert the key into the hole switch and turn clockwise on the power before driving. If the power switch design is the rocker switch, please press the “—” and then you will turn on the power.

Please turn the key by anticlockwise direction and remove the key from the power switch after using. Or press the “0” and then you can turn off the power.

S2043 key switch-6a

S2043 rocker switch-6b

**WARNING:** If the key is removed during use or press the rocker switch to “0”. The electromagnetic brake will be locked and your scooter will stop suddenly.

#### ››› Speed controller

Speed controller used for the forward and back of scooter.

- Hold the U-shaped handlebar.
- The right forefinger handle moves your scooter in a forward direction. (FIG-7a)
- The left forefinger handle moves your scooter in a back direction. (FIG-7b)
- The speed control will return to the center position when release it and the brake will be locked.

»» **Battery power indicator(FIG-8)**

Battery power indicator is an instrument that indicates the status of charging in your batteries so as to provide reference data of how long the scooter can run normally. There are three indicating lights with different color: red, yellow and green. When you turn on the power and set the rocker switch or key switch to 「-」, the battery power indicator will indicate the status of charging in your batteries.

FIG-8

»» **Unfolding and folding body**

Unfolding:

1. Press the release switch inward(FIG.9)
2. The frame will loosen automatic and use one hand pull the drawtube forward, then another hand press the carry handle downward until balance.(FIG.10)
3. Press down the seat cushion and unfold seat back in place.(FIG. 11)

Press the release switch to the drawtube's direction

FIG-9

Pull the drawtube forward

FIG-10

Press the carry handle downward

Unfold the seat back

FIG-11

Press down the seat cushion until balance

Folding:1.Push the seat folding switch inward which is under the seat (FIG-12) Use one hand push inward and the other hand press down the seat back margin 2. The front and back frame will draw close automatic and then pull the carry handle (FIG-13), 3. When the frame fold to some degree, use your both hands (One hand press the lamp and the other hand press the seat back margin) push inward (FIG-14), locking in place and trap inward until the frame fold totally.

Press the seat margin and fold the seat back

Push the seat folding switch inward

FIG-12

Pull the carry handle upward until the frame fold

FIG-13

One hand press the lamp and the other hand push the seat margin inward

**Warning:**

1. Please keep your fingers away from the spots because that may clip your fingers when unfolding the scooter body,
2. Please don't unfold the scooter body before the lock lever has been turned on.
3. When adjusting the seat, the seat reset after you hear the "click" sound.

»» Telescopic handlebar

Mobile Press down handlebar switch and adjust the handtube to the extreme, then unlock and loosen the quick release(FIG-15a,15b)

FIG-15a

FIG-15b

◆ Geniescooter

»» Control panel

The control panel is an operating controlling component, which is mainly to control power supply's connection and cut off, moving forward and backward as well as show the battery level, including power switch, speed controller switch, battery power indicator, control panel, cover, etc. (Genie FIG-16)

S2061 control panel FIG-16

- ① Power indicator
- ② Speed controller
- ③ U-shaped
- ④ Rocker switch
- ⑤ Cover

»» Power switch

Genie rocker switch (FIG-17)

Press the rocker switch to “-” before dive and then you will turn on the power. Press the rocker switch to “0” after drive and then you will turn on the power.

S2061 rocker switch FIG-17

»» Speed controller

»» Forward and back control as below :

- Hold the U-shaped handlebar
- The right forefinger handle moves your scooter in a forward direction. (FIG-18a)
- The left forefinger handle moves your scooter in a back direction. (FIG-18b)
- The speed control will return to the center position when release it and brake will be locked.

FIG-18a

FIG-18b

»» Battery power indicator(FIG-19)

Battery power indicator is an instrument that indicates the status of charging in your batteries so as to provide reference data of how long the scooter can run normally. There are three indicating lights with different color: red, yellow and green. When you turn on the power and set the rocker switch or key switch to “-”, the battery power indicator will indicate the status of charging in your batteries. Green part means the power is sufficient, yellow part means the power is weak, red part means you need to recharge your battery in time.

FIG-19

### >>> Unfolding and folding body

Genie Unfolding : 1. Use your hand to press outward the release switch which is on the handle tube (FIG-20) , then use the other hand to hold the edge of the seat, 2. Pull outward of the scooter body with your two hands. (FIG-21) 3. When the body has been unfolded to some degree, use one hand to push down the carry handle until balance (FIG-22) Unfold the seat back in place (FIG-23) .

FIG-20

Press the release switch (Open the release switch)

FIG-21

1. One hand hold the handlebar while the other hand hold the seat  
2. Pull outside

FIG-22

Push down the floor panel with your hand until the scooter body is horizontal

FIG-23

Rise up the back rest

### >>> Telescope handlebar

Solax-Genie folding instruction:

1. Push the seat fold button inside (FIG 24).
2. Push down the seat during scooter body folding (FIG 25).
3. Collect scooter body with your two hands, until it fold totally (FIG 26).

FIG-24

Push the seat fold button inside

FIG-25

Push down the seat during scooter body folding

FIG-26

Collect scooter body with your two hands, until it fold totally

#### Warning:

1. Please keep your fingers away from the spots because may clip your fingers when unfolding the scooter body.
2. Please don't unfold the scooter body before the lock switch has been turned on.
3. When adjusting the seat, the seat reset after you hear the "click" sound.

### >>> Controller

Controller is fixed in the rear cover and it receives the signal from the control panel and transfers to motor, brake and bulb.

#### Warning:

Please do not place the controller in the humid environment. If the controller has been affected by moisture, make sure to dry it before operation

»» Brake

SOLAX-scooter is equipped with electromagnetic brake. Please do not switch off the brake by hands. Please raise the head of the scooter or fold your scooter to pull when you need to move your scooter by hands.

»» Motor/ Transmission component

Motor/Transmission components will convert the electrical energy to rear wheel drive.

## Specification

Model	Mobie/Genie
Overall Length:	930mm (Unfold)
Overall Width:	50mm
Tires (Solid wheels):	Tire Size Front 6 "x 1.5" Tire Size Rear 7 "x 2.36 "
Maximum Speed:	6Km/H
Safe Gradient/ Maximum Gradient :	0-12°
Range Up To:	15Km
Turning Circle:	1.35M
Weight (including battery pack):	26.6KG
Color:	White, Black, Red , Blue
Seat :	Low on, no angle is adjus table
Brake:	Intelligent, regenerative, and electromagnetic
Drive system :	Rear wheels' drive .(4 5A)
Handlebar:	2 adjustable parts
Maximum weight capacity:	125kg(275lb)
Obstacle height :	60MM
Battery :	Lithium battery
Motor:	120W
Battery Charger:	DC24V 2A(OUTPUT: 29.4V 2A)

## Safety Requirements

### Driving Surface

SOLAX-scooter has the best stability under the normal driving conditions (such as dry level ground with the constituent concrete or asphalt). However, it may meet other grounds, such as grassland, gravel and etc. So there are some requirements before riding the scooter:

- Please read this manual before driving the scooter.
- Make sure about your physical condition before driving the scooter.
- SOLAX scooter can be used in the park and grass safely.
- Try to avoid driving in the loose gravel, covered soil and sandstone.
- Do not drive the scooter if you can't tell the condition of the roads.
- SOLAX scooter can stride over an obstacle with a certain height, but you must minimize the speed and slowly drive over the obstacle.(FIG 27)

Load capacity	Height
≤ 125kg	60mm

FIG-27

### Warning:

Do not exceed the specify climbing angle (0-12°) Please do follow the manual instruction before operation.

### Safety precautions

- Do not operate your scooter before you fully read and understand this manual.
- Do not carry passengers or drunk driving when you drive the scooter.
- Remove the key when you leave the scooter.
- Ensure the seat is fixed and fastened.
- Please slow down when driving on the uneven or soft ground.
- Please slow down before turning.
- Do not park on slopes.
- Approach slope end, slopes, raised group and unprotected edge zone(such as curbside, vestibule and stairs ets.) with extra care.
- Changing initial setting or refitting your scooter is forbidden.
- Please be careful when driving in the busy streets, market or shopping center; driving in the unsafe zone. Prohibited area is forbidden. Please consult local authorities about the traffic rules for scooters.

- Hold the handle with both hands and put both feet on the footboard when driving.
- Using your scooter as a seat on moving vehicle is forbidden.
- Please ensure the safety of the batteries when your scooter is on the other vehicles for transportation.
- Do not drive scooter after drinking.
- Climbing or driving along the edges of roads is forbidden, otherwise, the scooter will be damaged permanently.
- Do not exceed the maximum tolerance gradient when driving.
- Do not carry passengers or exceed the maximum weight capacity.
- Do not reverse on uneven slopes or uneven ground; be careful when passing over slopes.

## Battery and Battery Charging

This scooter is designed with one maintenance free deep discharging Li-polymer battery with a long usage time as well as a long useful life. Charging with a 24V/2A charger makes it easy to use.

- Charge the battery before the first operation. Please charge the battery every 3

### The guide for charging steps:

1. Close (the Rocker switch or key switch) and place the charger. Then switch off its power and open the bottom charger cap and insert the plug into the charge socket. FIG-28

FIG-28a

FIG-28b

Cannon plug

**Warning :**  
Match the charger socket and charger plug properly.

2. Insert one end of the AC power wire into the socket of household power then insert the other plug into the socket of the charger.(FIG 29) Verify the connection before you turn on the main power switch. Red light indicates power on, orange light indicates on charging and green light means fully charged.

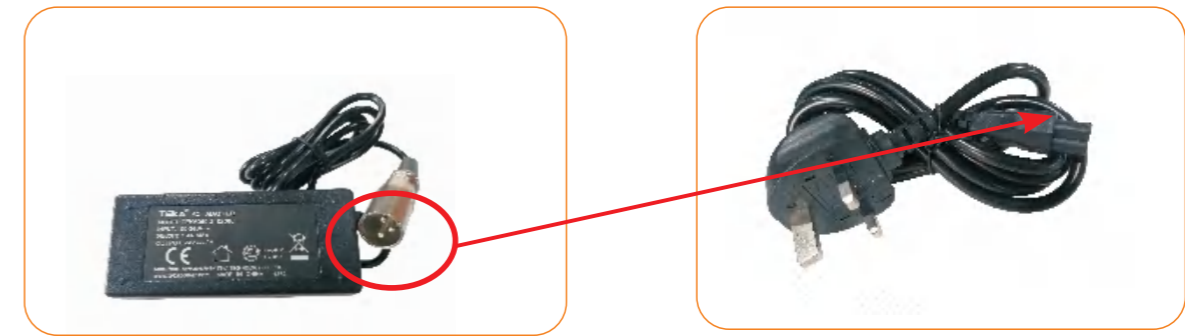
FIG-29

3. Turn off the power switch first when the battery is fully charged. Then take out the DC charging connector before you take out the AC input power socket.

**Warning :**  
You must turn off the power switch once the battery is fully charged and then take out the plug otherwise you may shorten the working life of the battery.

4. If the red light is off when the power on, please check whether the power plug is properly inserted.

5. Normally allows 8-14 hours for charging.

**Note :** The scooter has a self-locking function to prevent driving when you charge the battery.

### Guide to safe and lasting battery

#### How does charger work?

When the battery voltage is low, the battery charger outputs a large current to charge the battery. When battery voltage is close to full, the battery charger outputs a small current. When battery is full, the battery charger will output very little current which is almost zero. Therefore, The battery will continually charge after connecting the charge, but will not overcharge. It is better to charge no more than 24 hours.

#### How does the indicator light(LED) in charger display.

There are two LED indicator lights in the charger. The red indicator light is the power indicator light. Regarding the other one, it is orange when charging and it will become green when the battery is charged fully. The red one will sometimes still on after pulling the charger plug out from the power socket; then you may think there is some thing wrong with the charger, however, in fact, it needs seconds for the red light to snuff out when the battery voltage is up to 26V.

## Other chargers can be used?

In order to charge safely and efficiently, we suggest using the charger supplied by original manufacturer.

## How often should charge the battery?

The charge time depend on the following elements:

- Driving scooter all day.
- Driving scooter occasionally for few times.

### Following information will offer the safe and reliable charge method.

- If you driving scooter everyday, you should charge it after using. So that it is convenient for you to use it next day. Charging of the battery should be vary from 8 up to 12 hours.
- If driving scooter once a week, then charge battery once a week. Charging the battery should be vary from 12 up to 14 hours.
- Make sure your battery fully charged and saturation.

### How can you get the maximum operating distance ?

Some driving conditions such as hills, sidewalk gaps, uneven and soft surface, turning and against the wind will affect the driving distance and time after per battery charging.

### Several methods for obtaining the maximum driving distance

- Fully charged your battery before driving it.
- Avoid hills, gaps, macadam and soft surface.
- Only to carry the necessities and reduce the luggage weight.
- Keep a constant speed. Avoid intermittent driving.

### Warning:

Please do not dismantle the Li-polymer battery and do not add water. Failure to observe this note will invalidate the warranty, and damage the battery and scooter

## Why the power of the new battery is weak?

Deep-cycle battery uses the unique chemical technology and design. It can be charged quickly and can be used for a long time after full-charged. The battery is fully charged already before leaving the factory, but may change its initial performance during the transportation due to the temperature. The power of the battery will lose in high temperature and it will extend the charging time in low temperature. The battery needs few days to adapt to the surrounding environment before it gets stable performance.

More importantly, deep-cycle battery can only get high performance after using for several times of charging-discharging cycles.

### Please follow the steps to improve the battery performance as below:

1. New battery must be fully charged before using and it can reach 88% capacity.
2. Please drive in the safe and the familiar area. Low speed is recommended at the first time. Do not travel too far until you are familiar with the operation and the battery should be fully charged.
3. Please give the battery another fully charging after operated it in the second times and this will make the battery up to 90% capacity.
4. Send time to drive scooter.
5. The battery capacity will top to 100% and extend the driving time after four to five charging and discharging cycles.

## How to ensure the battery life?

A fully charged battery will provide good performance and extend battery life, so keep the battery full charged whenever possible. It will affect performance and shorten battery life if you over discharging, seldom charging or do not fully charge.

## How to store scooter and battery?

### If you don't operate your scooter, please follow the instructions:

- Charge your battery fully before you store it.
- Disconnect the battery from controller attachment joint.
- Store your scooter in a dry and warm place.
- Avoid to store scooter in temperature variation place.

### Warning:

If battery was frost, it should be kept warming for several days before charging.

**If you will leave the scooter for a long time, please lay a plate under the footboard to support scooter. It can reduce stains of tires when wheels pressing on the ground.**

## Operational Instruction

### Inspection before operating:

- Please inspect the battery is fully charged or not. (Refers to “V、 Battery and Battery Charging)
- Please make sure you know the route well. (Including the crowd, animal, obstacle)
- Avoid the un-flat and slope road condition.
- Check if the key is inserted into the key switch or the end of rocker switch is pressed. (Refers to “II、 Function Introduction”)
- Make sure your hands are steering the handle bar. (Refers to “II、 Function Introduction”)
- Please check the horn is working or not.
- Please inspect the seat is on right position and locked.
- Please check the tiller quick release is locked or not.

### Driving

Please follow the instructions after you choose your route:

- Press the speed control lever gently by your right thumb.
- Brake releases automatically, and scooter will move forward stable with the speed you set.
- Push right handlebar moves scooter turn left when operating.
- Push left handlebar moves scooter turn right when operating.
- Steer the handlebar in the middle, scooter will move in forward direction.
- Release the speed control lever, until scooter has slowed down or fully stopped.
- Press the speed control lever gently by your left thumb, then scooter will backstable with the speed you set.
- Brake can be automatic locked when scooter fully stopped.

### Finish operating

1. Make scooter fully stopped
2. Off the switch and remove the key or press the rocker switch “0”.
3. Get down scooter in safety.

## Basic Troubleshooting

The main reason is battery without sufficient capacity or performance declined, and most of the electrical equipment issue can be solved by yourself.

- Please check the key is inserted into the switch /rocker switch, and keep it “on”.
- Check it if the battery is fully charged or not. (Refers to “VI”、 Battery and Battery Charging)
- If the battery capacity is insufficient, please increase the charging time/ cycles. (Refers to “VI、 Battery and Battery Charging)
- If the issue continued, then battery capacity testing is needed.

### Scooter can not work continually when you are driving it

- Battery line is loose.
  - Motor carton bush is damaged.
- Speed suddenly slows down when driving
- Battery capacity is insufficient.
  - Battery aging.

If met the problem that you can not solve it by yourself, please contact your dealer for further consult and assistance.

## Maintenance

There are few parts of SOLAX foldable scooter require maintenance.  
The following issues must be checked and maintained periodically:

### The connection of the battery and electrodes

- Ensure the electrode connections are tight and have no corrosion.
- The battery should be placed flatly inside the battery holder. Wire Connector
- Check all the wire connectors regularly.
- Check all the wiring insulation condition, including the plug of the charger regularly
- Repair or change the damaged connector and connector joint.

### ABS Plastic Cover

● Control panel, front cover, footboard and back cover are all made by durable ABS plastic with baking finish on the surface. Do not use oil or other chemical liquids to wipe the scooter. In order to prevent the electrical components from damage, do not wash the scooter from the tap directly.

### Bearings and Motor/Transmission Parts

- The components have injected lubricant and are sealed; therefore, it is unnecessary to inject lubricant any more.
- Protect all the electronic components from moisture, such as control panel, battery charger and other electric controlled components.
- If some components become damp, please dry it before use.

### Storage

Refers to “V”、 Battery and Battery Charging and How to handle scooter and battery

# SOLAX ELECTRIC SCOOTER

## Warranty Card

User name		ID No.	
Address		Phone No.	
Model		Item No.	
Purchasing Date		YY	MM DD
Guarantee period	13 months warranty from purchasing date (6 months warranty for battery)		
Manufacturer	Dongguan Prestige Sporting Goods Co. Ltd.		
Dealer (Stamp/Signature)			
Phone No./Address			

### (1). Warranty Statement :

Dealer is responsible for the repair or replacement from the parts when quality problems of Materials, manufacture or assembling comes out under the normal usage.

### (2). Warranty Exclusions:

- Damages caused by improper operation and maintenance;
- Damages caused by irregular spare parts;
- Damages caused by accidents;
- Consumables are not covered by warranty such as the tires, bearings, bulbs etc.
- Damages caused by unauthorized refit of the scooter;
- Damages caused by natural disasters such as hurricanes, floods and earthquakes;

### 1.Warranty stub for dealer

# SOLAX ELECTRIC SCOOTER

## Warranty Card

User name		ID No.	
Address		Phone No.	
Model		Item No.	
Purchasing Date		YY	MM DD
Guarantee period	13 months warranty from purchasing date (6 months warranty for battery)		
Manufacturer	Dongguan Prestige Sporting Goods Co. Ltd.		
Dealer (Stamp/Signature)			
Phone No./Address			

**(1). Warranty Statement :**

Dealer is responsible for the repair or replacement from the parts when quality problems of Materials, manufacture or assembling comes out under the normal usage.

**(2). Warranty Exclusions:**

- a. Damages caused by improper operation and maintenance;
- b. Damages caused by irregular spare parts;
- c. Damages caused by accidents;
- d. Consumables are not covered by warranty such as the tires, bearings, bulbs etc.
- e. Damages caused by unauthorized refit of the scooter;
- f. Damages caused by natural disasters such as hurricanes, floods and earthquakes;

**2.Warranty stub for customer**

## SOLAX ELECTRIC SCOOTER

**Warranty Card**

User name		ID No.	
Address		Phone No.	
Model		Item No.	
Purchasing Date	YY MM DD		
Guarantee period	13 months warranty from purchasing date (6 months warranty for battery)		
Manufacturer	Dongguan Prestige Sporting Goods Co. Ltd.		
Dealer (Stamp/Signature)			
Phone No./Address			

**(1). Warranty Statement :**

Dealer is responsible for the repair or replacement from the parts when quality problems of Materials, manufacture or assembling comes out under the normal usage.

**(2). Warranty Exclusions:**

- a. Damages caused by improper operation and maintenance;
- b. Damages caused by irregular spare parts;
- c. Damages caused by accidents;
- d. Consumables are not covered by warranty such as the tires, bearings, bulbs etc.
- e. Damages caused by unauthorized refit of the scooter;
- f. Damages caused by natural disasters such as hurricanes, floods and earthquakes;

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